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MANUFACTURER: BIOTECH MEDICAL SIMULATION SL

ZAFRA 06300 (Badajoz)

C.I.F.: B06694533

www.biotme.com
[@BIOTME](#)

GENERAL DATA

Trade name: **BIOT-HD6**

Full name: **Dermatological ultrasound simulator for performing ultrasound-guided punctures, infiltrations, and biopsies with a thick needle, etc.**

Catalogue: BIOTME

Family: BIOT-H DERMATOLOGY

Item Type: Enables ultrasound-guided punctures, infiltrations, and biopsies with thick needles, etc.

DESCRIPTION

It allows ultrasound-guided punctures, infiltrations, and biopsies with a thick needle, etc.

It allows the following procedures to be performed:

- 1 Fine needle aspiration for fluid collection.
- 2 Biopsy with a thick needle of subcutaneous nodules and training to differentiate lesions suggestive of malignancy from healthy lymph nodes.
- 3 Examination and infiltration of hidradenitis suppurativa fistulas.
- 4 Localisation and removal of foreign bodies.
- 5 Tumour delineation.
- 6 Ultrasound-guided deep infiltration on bone plane and identification of cranial openings.

BOX CONTENTS:

- Simulator BIOT-HD6
- Technical Data Sheet

CONSERVATION

Store refrigerated between 4°C and 8°C.

MAINTENANCE

Apply plenty of ultrasound gel over the convex surface with rounded edges. Remove the vacuum packaging by trimming the four sides. The arrangement of the objects is random. Once the activity is completed, we recommend removing the top and bottom silicone layers. The contents of the Dermofantoma are disposable

CONDITIONS OF USE

Humidity:

Transpotation: <80% HR

Storage: <80% HR

Working: <80% HR

Temperature:

Working: (>04C <8°C (>32°F <77°F)

Transpotation: (4°C<8°C) 39°F <44,6°F)

Storage: (>4°C <25°C)(>32°F <45,6°F)

PRECAUTIONS

Exercise caution when pressing the simulator with any object that could damage it. Do not subject it to conditions exceeding those indicated. For professional use only and only valid as a defined trainer. Keep out of sight and reach of children.

WARRANTY

TFour-month warranty. Perishable simulator. Keep at the indicated temperature and maintain the vacuum seal. If you notice that it does not retain the vacuum when you unpack it, notify the manufacturer. Once the vacuum is lost, it is considered used. Only manufacturing defects unrelated to the loss of vacuum and/or storage conditions will be covered

RECYCLING

"Once its useful life has ended, the contents of the Dermofantoma should be disposed of in an organic waste container. The silicone layers must be properly managed in accordance with the regulations of the country of use."

PRODUCT PROPERTIES

Boiling point: 100°C

Melting point: not specified

density: 1,1

Appearance and odor: solid and odorless

Solubility in water: not soluble

Flash point: non-flammable

HEALTH EFFECTS

Skin: No irritation is expected upon contact, as it has a very similar composition.

Eyes: No irritation is expected upon contact, as it has a very similar composition.

Inhalation: This material has a very low vapor pressure, and no inhalation hazard is expected.

Health hazards: It does NOT present health risks from ingestion or inhalation in small quantities.

EMERGENCIES AND FIRST AID

kin contact: If the skin comes into contact with the molten material, treat as a common burn. This material does not irritate the skin.

Inhalation: The product is not expected to irritate the nose, throat, or respiratory tract.

Ingestion: The product has a low order of acute toxicity.

FIRE-FIGHTING MEASURES

SSpecific hazards: It is NOT a flammable material.


KEEP OUT OF SIGHT AND REACH OF
CHILDREN

**ALL SIMULATORS
MANUFACTURED BY BIOTECH
MEDICAL SIMULATION SL
ARE FREE FROM MATERIALS
OF ANIMAL ORIGIN.**

FOR PROFESSIONAL USE ONLY.
Non-medical materials. Simulator
for educational use.

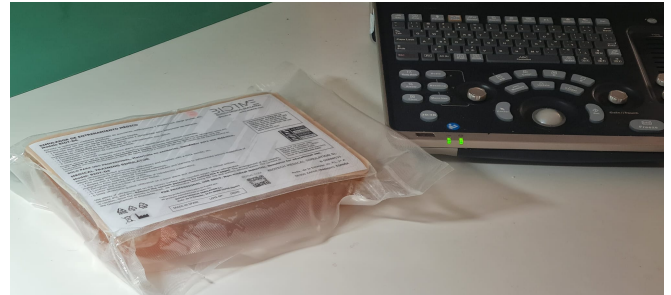Additional information and
warranty conditions at
www.biotme.com

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